

CUSTOMIZED CONTOUR

ADD VOLUME TO CHEEKS FOR LIFT
AND SHAPE THE CHIN FOR CONTOUR



JUVÉDERM®
VOLUMA™ XC

Actual patient treated with JUVÉDERM® VOLUMA™ XC
in the cheek and chin area. Results may vary.

JUVÉDERM® VOLUMA™ XC Injectable Gel Important Information

APPROVED USES

JUVÉDERM® VOLUMA™ XC injectable gel is for deep injection in the cheek area to correct age-related volume loss and for augmentation of the chin region to improve the chin profile in adults over 21.

Please see Important Safety Information on following pages.

A CLOSER LOOK AT CONTOURS



JUVÉDERM® VOLUMA™ XC

AWARD FOR BEST CHIN FILLER

JUVÉDERM® VOLUMA™ XC IS THE FIRST
AND ONLY FDA-APPROVED CHIN FILLER



WOMEN The female face has an oval shape, with more defined cheeks and a contoured chin.



MEN The male face has a square shape, with wider cheeks and a more well-defined chin.

YOUR CHEEKS CAN PORTRAY A MORE YOUTHFUL LOOK*

- Are you hoping to achieve a lifted, more youthful look in your cheeks?
- Have you noticed your cheeks starting to flatten?

Temporarily adding volume with JUVÉDERM® VOLUMA™ XC can help contour the cheeks, leading to a natural-looking, more youthful appearance.

*Patients perceived themselves as looking 5 years younger at 6 months after treatment and 3 years younger at 2 years after treatment, as reported by people in the clinical study. Results may vary.

IMPORTANT SAFETY INFORMATION

Are there any reasons why I should not receive JUVÉDERM® VOLUMA™ XC?

Do not use if you have a history of multiple severe allergies or severe allergic reactions (anaphylaxis), or if you are allergic to lidocaine or the Gram-positive bacterial proteins used in this product.

What precautions should my doctor advise me about?

- Minimize strenuous exercise and exposure to extensive sun or heat within the first 24 hours following treatment. Exposure to any of these may cause temporary redness, swelling, and/or itching at the injection site

YOUR CHIN IS A KEY PART OF YOUR FACIAL APPEARANCE

- Do you feel like your chin looks smaller, especially when looking in profile view?
- Do you feel that your chin doesn't quite suit your face?

Shaping your chin with JUVÉDERM® VOLUMA™ XC can help you achieve a natural-looking contour.

- Tell your doctor if you are pregnant or breastfeeding. The safety of this product for use during pregnancy or while breastfeeding has not been studied
- The safety of JUVÉDERM® VOLUMA™ XC has not been studied in patients under 35 years or over 65 years for cheek augmentation, or under 22 years and over 80 years for chin augmentation

Please see additional Important Safety Information on following pages.

JUVÉDERM® VOLUMA™ XC
IS THE ONLY FDA-APPROVED
FILLER FOR
THE CHIN

HER GOAL
NOTICEABLE CONTOUR
IN HER CHIN



BEFORE



AFTER

ASHLEY AGE 32
AERIAL YOGA INSTRUCTOR

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 3.3 mL of JUVÉDERM® VOLUMA™ XC was injected into the chin area.

The primary endpoint in the clinical study was not met in patients with darker skin types. However, their satisfaction was still high.

IMPORTANT SAFETY INFORMATION (continued)

What precautions should my doctor advise me about? (continued)

- JUVÉDERM® VOLUMA™ XC is intended for use in the cheek or chin areas. The safety and effectiveness for treatment in other areas have not been established in clinical studies
- Tell your doctor if you have a history of excessive scarring (thick, hard scars) or pigmentation disorders. The safety of JUVÉDERM® VOLUMA™ XC has not been studied in these patients and may result in additional scars or changes in pigmentation

- Tell your doctor if you are on therapy used to decrease the body's immune response (immunosuppressive therapy). Use may result in an increased risk of infection
- Tell your doctor before treatment if you are using substances that can prolong bleeding, such as aspirin, ibuprofen, or other blood thinners. As with any injection, this may result in increased bruising or bleeding at the injection site

Please see additional Important Safety Information on following pages.

JUVÉDERM® VOLUMA™ XC IS
THE #1 CHOSEN
DERMAL FILLER
IN THE JUVÉDERM® COLLECTION

HIS GOAL

SHAPE HIS CHIN FOR A
MORE WELL-DEFINED PROFILE



BEFORE



AFTER

DYLAN AGE 30
USMC VETERAN

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 3.3 mL of JUVÉDERM® VOLUMA™ XC was injected into the chin area.

The primary endpoint in the clinical study was not met in men. However, their satisfaction was still high.

IMPORTANT SAFETY INFORMATION (continued)

What precautions should my doctor advise me about? (continued)

- Patients who experience skin injury near the site of injection may be at a higher risk for adverse events
- JUVÉDERM® VOLUMA™ XC was not studied in patients with significant loose skin of the chin, neck or jaw

Please see additional Important Safety Information on following pages.

HER GOAL
CREATE A SMOOTH, NATURAL-
LOOKING CONTOUR IN HER CHEEKS



LESLIE AGE 39
EDUCATION POLICY ADVOCATE

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 4.0 mL of JUVÉDERM® VOLUMA™ XC was injected into the cheek area.

*JUVÉDERM® VOLUMA™ XC has been shown to last up to 2 years in the cheeks and up to 1 year in the chin with optimal treatment.

JUVÉDERM® VOLUMA™ XC IS THE
LONGEST-
LASTING PRODUCT
OF ITS KIND*



IMPORTANT SAFETY INFORMATION (continued)

What precautions should my doctor advise me about (continued)

- The effect of JUVÉDERM® VOLUMA™ XC injection into the chin on facial hair growth has not been studied

Please see additional Important Safety Information on following pages.

ADD VOLUME TO
YOUR CHEEKS FOR
A MORE
YOUTHFUL LOOK*

HER GOAL
ACHIEVE A MORE YOUTHFUL
LOOK IN HER CHEEKS



BEFORE



AFTER

EVA AGE 49
WRITER

IMPORTANT SAFETY INFORMATION (continued)

What are possible side effects?

The most common reported side effects with JUVÉDERM® VOLUMA™ XC included redness, swelling, pain, tenderness, firmness, lumps/bumps, bruising, discoloration, and itching. Most side effects resolved within 2 to 4 weeks. These side effects are consistent with other facial injection procedures.

Please see additional Important Safety Information on following pages.

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 2.3 mL of JUVÉDERM® VOLUMA™ XC was injected into the cheek area.

*Patients perceived themselves as looking 5 years younger at 6 months after treatment and 3 years younger at 2 years after treatment, as reported by people in the clinical study. Results may vary.

ACHIEVE THE CUSTOM CONTOUR YOU DESIRE WITH JUVÉDERM® VOLUMA™ XC



HER GOAL

VOLUMIZE HER CHEEKS FOR LIFT AND
HER CHIN FOR IMPROVED CONTOUR

BEFORE



AFTER



AVI AGE 39

PROFESSIONAL TV JOURNALIST

IMPORTANT SAFETY INFORMATION (continued)

What are possible side effects? (continued)

Most side effects will resolve with time. Your doctor may choose to treat side effects persisting over 30 days with antibiotics, steroids, or hyaluronidase (an enzyme that breaks down hyaluronic acid).

Please see additional Important Safety Information on following pages.

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 5.4 mL of JUVÉDERM® VOLUMA™ XC was injected into the cheek area, and a total of 2.3 mL was injected into the chin area.

HER GOAL

VOLUMIZE AND CONTOUR HER CHEEKS; SHAPE HER CHIN TO BETTER SUIT HER FACE

1 PRODUCT
2 FDA-APPROVED
USES

BEFORE

AFTER

MONA AGE 31 JEWELRY DESIGNER

Actual patient. Results may vary. Unretouched photos taken before treatment and 1 month after treatment. A total of 2.4 mL of JUVÉDERM® VOLUMA™ XC was injected into the cheek area, and a total of 2.1 mL was injected into the chin area.

IMPORTANT SAFETY INFORMATION (continued)

What are possible side effects? (continued)

One of the risks with this product is unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious and may be permanent. These

complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin.

As with all skin injection procedures, there is a risk of infection.

Please see additional Important Safety Information on following page.

JUVÉDERM® VOLUMA™ XC
MAY BE RIGHT
FOR YOU **IF:**



YOU HAVE
SEEN YOUR
CHEEKS
FLATTEN
OVER TIME

YOU WANT
TO CONTOUR
YOUR CHIN FOR
OVERALL AESTHETIC
IMPROVEMENT
IN THE CHIN AREA



YOU WANT TO
SHAPE
AND
DEFINE
YOUR CHIN



Actual patients treated with JUVÉDERM® VOLUMA™ XC in the cheek and chin area.
Results may vary.

JUVÉDERM[®] PRO CONTOUR

WITH JUVÉDERM[®] VOLUMA[™] XC

AllēSM

YOUR PREMIER AESTHETICS REWARDS PROGRAM



JOIN & EXPLORE

Become a member in a few simple steps. Create a new account or use your existing *Brilliant Distinctions*[®] info.



TREAT & EARN

Earn points on Allergan Aesthetics products, including JUVÉDERM[®] VOLUMA[™] XC, and a variety of other in-office treatments—from facial injectables to microdermabrasion facials.



REDEEM & SAVE

Your points, your way. Either use them as you go, or store points in your Wallet for big savings later.

IMPORTANT SAFETY INFORMATION (continued)

Visit Juvederm.com or talk to your doctor for more information. To report a side effect with JUVÉDERM[®] VOLUMA[™] XC, please call Allergan at 1-800-433-8871.

JUVÉDERM[®] VOLUMA[™] XC is available only by a licensed physician or properly licensed practitioner.

JUVÉDERM® PRO CONTOUR

WITH JUVÉDERM® VOLUMA™ XC

JUVÉDERM® VOLUMA™ XC
THE #1 CHOSEN
DERMAL FILLER
IN THE JUVÉDERM® COLLECTION



ASK YOUR AESTHETIC SPECIALIST ABOUT
JUVÉDERM® VOLUMA™ XC AND HOW YOU CAN ACHIEVE
A CUSTOMIZED CONTOUR IN THE CHEEKS AND CHIN.

Allergan
Aesthetics
an AbbVie company

AllēSM is a service mark of Allergan, Inc, an AbbVie company.
JUVÉDERM®, VOLUMA™, and their designs are trademarks of
Allergan Holdings France SAS, an AbbVie company, or its affiliates.
© 2020 AbbVie. All rights reserved. Juvederm.com
Re-order: JUV142688 12/20 007750